

Universal Ankle Compressive Cryotherapy Device

DESIGNED TO

- Immobilizing the ankle
- Fighting against pain
- Limiting bleeding and hematoma
- Managing post operative edema
- Easy to use at home


Excell' ICE

Excell'ICE cold kits combine cryotherapy with modulated static compression using an inflation pump.

Easy to apply, its technology consists of alternating compressive cold during recovery and/or healing sessions, commonly recommended in the days following an injury or post-operation.

It is well known that external static compression with ice increases the extent of cooling at the surface of the skin.

A unique new concept where the compression chamber is directly integrated into the cold pack, making it much easier to use. This new technology makes it easier to carry out 20-30 minute cryotherapy-compressive recovery sessions. These portable devices are very practical to use at home or when travelling.

For athletes, this will ease musculoskeletal pain immediately after their matches or competitions and enable them to recover optimally, thereby reducing the time they spend out of competition due to pain, injury, fatigue, etc.



Recommended use:

20 min. by session every 4 hours
First session in Operating Room after dressing

Made in Europe

INDICATIONS

- Post-operative care
- Analgesic treatment
- Sport trauma (sprain, tear, etc..)
- After rehabilitation session

CONTRAINDICATIONS

- Allergy to cold
- Insensitivity to cold
- Raynaud's syndrome
- Cryoglobulinemia
- Arteritis
- Diabetes
- Polyarthritis...

Excell' ICE

Today's most user-friendly system: "Cold Pack" and "Compression Chamber" combined in a single device.

- Compression multiplies the effects of cold
- Cold packs remain flexible to adapt to body shape even at -4°F (-20°C)
- No frost to avoid skin burns
- Hypoallergenic Lycra with tightened mesh (included)
- 360° coverage around the Wrist in a single cold pack
- Freezer bag with zip slider for easy opening/closing (included)

Ankle Cryo kit

Introducing the Ankle Cryo Kit, an ankle cooling solution that is incredibly easy to apply. With its universal one-size-fits-all design, it will fit any ankle. This kit includes one single cold pack surrounding the ankle and an easy Velcro, providing comprehensive coverage for your ankle. The innovative design combines cold therapy with compression in a single, convenient piece. Achieve optimal compression using the included manual pump.



1 Place the foot in the cold pack. The heel must be in the hole.



2 Fold down the left part of the cold pack on the foot, and take the velcro of the right side.



3 Fold down the right part of the cold pack on the left side and fix the velcro.



4 Turn the tap to O and screw the pump.



5 Inflate (do not exceed 10 pressures). Turn the tap to F and unscrew the pump.

Universal Ankle Compressive Cryotherapy Device

Ref. ORT40PACK-CH



Cold pack



Manual pump



Lycra



Freezer bag



Suitcase